**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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 english

## Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

## Intended purpose

SacroLoc is a medical device. It is an orthosis<sup>1</sup> for the stabilization of the pelvis and targeted relief of the sacroiliac joints.

## Indications

- Sacroiliac joint disorders (SI joint syndrome, osteoarthritis of the SI joint, SI joint instability)
- Pelvic girdle instability, e.g. symphysis rupture and loosening
- Sacroiliac joint blockage (acute and prophylactic)
- Myalgia and tendinopathy in the pelvic region
- Structural disturbances following spinal fusion (in the transition from L5 to S1)
- Non-surgical treatment following pelvic fractures

## Risks of using this product

### Caution\*

**Please closely observe the specifications in these instructions for use and the instructions given by the specialist.**

- SacroLoc must only be worn in accordance with the specifications contained in these instructions and used for the areas of application (places of use) listed.
- No product liability is accepted in the event of improper use.
- The suitability and use of the orthosis<sup>1</sup> during pregnancy is to be determined by a physician on a case-by-case basis. If wearing the orthosis<sup>1</sup> during pregnancy feels uncomfortable, check size adjustments and application technique, seek medical advice, and refrain from further use if necessary. Side effects for the mother and fetus are not known.
- Only a specialist<sup>2</sup> may provide instruction.
- Use in combination with other products, e.g. as part of compression treatment (medical compression stockings), must be discussed beforehand with your physician.
- The SacroLoc is to be worn under your T-shirt or undershirt.
- If your symptoms get worse or you notice unusual changes to your body, please contact your physician without delay.
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Do not put the SacroLoc on too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. For this reason, check the fit of the

product if you feel unpleasant pressure and consult the specialist in this regard or for general questions about handling.

- **Caution:** When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.

## Contraindications

Hypersensitivity reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
- Impaired sensation and movement disorders, e.g. in the case of diabetes mellitus

## Area of use

A – Back pad (massage cushion)

B – Right half of the fastening with finger pockets

C – Left half of the fastening with finger pockets

D – Tensioning straps

## Putting the product on

**1** Position the orthosis<sup>1</sup> centrally on the pelvis, ensuring that the upper edge of the orthosis<sup>1</sup> is no higher than the upper edge of the iliac crest and below the waist. **2** Then slide your fingers downward into the finger pockets provided on the fastening and steadily pull the fastening forward. **3** Then press the left half of the fastening onto your stomach and position the right half of the fastening on top of the left half so that it can be fastened to it on the left-hand side. When fastening the two halves, pull your fingers out of the left-hand finger pocket first and then out of the right-hand one. **4** Then take hold of the two tensioning straps and pull them forward, steadily and at the same time, until the desired level of tension (stabilization) is achieved. **5** Then fasten the two tensioning straps close together on the large fastening.

## Care instructions

**Note:** Never expose the SacroLoc to direct heat (e.g. heater, sunlight, in your car, etc.)! This may cause damage to the material, which can impair the effectiveness of the SacroLoc.

The sewn-in label of the the SacroLoc, containing information about the product name, size, manufacturer, washing instructions and CE identification, is located on the inside of the knitted fabric.

If necessary, you can wash the SacroLoc by hand at 30 °C with a mild detergent. Leave the orthosis<sup>1</sup> to air dry.

## Part of the body this product is used for

In accordance with the indications (pelvis, sacroiliac joints). See purpose.

## Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

## Assembly and fitting instructions

The SacroLoc is supplied in a standard size and ready for use.

## Technical specifications / parameters

SacroLoc is made of a knitted fabric with firm tensioning straps and removable back pads. The back pads are attached with an unremovable terrycloth cover. The back pads can be supplied as spare parts.

## Notes on reuse

This product is intended to be fitted to one patient only.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the SacroLoc have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

## Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine

the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

### Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

### Disposal

You can dispose of the product in accordance with national legal requirements.

## Specialists<sup>2</sup>

### General notes

Check the optimal and individual fit of the SacroLoc after the first fitting. Adjust the position of the back pads against the pelvis so that the pads are lying with the inner recess at the posterior superior iliac spine. Practice the correct procedure for putting on the orthosis with the patient. The product must not be changed in any improper way. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.

### Material content

Polyamide (PA), Thermoplastic elastomer (SEBS), Polyurethane (PUR), Spandex (EL), Polypropylene (PP), Polyester (PES), Polyoxymethylene (POM), Cotton (CO), Polyethylene (PE), Acrylic Adhesive (A)

**MD**<sup>1</sup> – Medical Device
**UDI**<sup>1</sup> – DataMatrix Unique Device Identifier UDI

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\* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

<sup>1</sup> Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

<sup>2</sup>A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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